


## POC-W212 Appearance



Front-view

Rear-view

\* Please use only VESA compatible mounting.

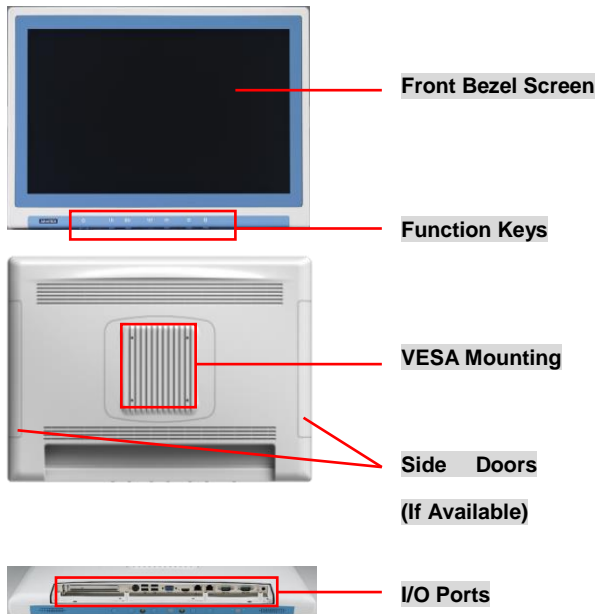

## Packing List


- (1) POC-W212 x 1
- (2) Accessory box x 1
  - ✓ This quick start manual x 1
  - ✓ Simpro HPU101-107 (18Vdc/5.55A) adapter x 1
  - ✓ Drivers & Manual CD x 1
  - ✓ Tool bag x 1: spare screws x 4, and DRAM thermal pad x 1.
  - ✓ Mylar x 1: for USB bracket fixture

## (1) Function Keys



- (1) Power On/Off
- (2) Volume Control
- (3) Touchscreen Function Control
- (4) Read Light Control
- (5) Brightness Control

## (2) I/O Ports

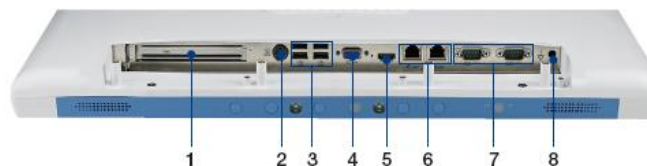

- (1) PCIe(x4) Card Slot
- (2) Power DC-IN
- (3) 2 x USB 2.0 Ports & 2 x USB 3.0 Ports
- (4) VGA Port
- (5) HDMI-out Port
- (6) 2 x Gigabit Ethernet (isolated) Interfaces (RJ-45)
- (7) 1 x RS-232 (isolated) & 1 x RS-232/422/485 (isolated) Serial Port
- (8) Equipotential Terminal Pin

## (3) Easy Setup

### Power On Computer:

- 1) Make sure Equipotential Terminal is connected to equipotential ground in the hospital.
- 2) Connect the adapter to Power DC-IN. Please note that using different adaptors may put the system into risk.
- 3) Press Power On/ Off. The button will turn green.

### Driver Installation:

Advantech supports “one key” driver installation. A user can simply double click an icon to install all the drivers.

- 1) Make sure the OS has been successfully installed. If you have difficulty installing an OS, please follow the instructions in the POC-W212 User Manual.
- 2) Download the “one key” package from the Advantech Support website:  
[http://support.advantech.com/support/new\\_default.aspx](http://support.advantech.com/support/new_default.aspx)  
 Open the folder and double click “InstallAll.exe.”
- 3) An Install dialog will appear. Follow the instructions shown in the dialog to finish the installation.

\* Please use a clean OS to install this auto installation; otherwise, unexpected errors may occur.  
 \* Should you only want to install specific drivers, please follow the instructions in the POC-W212 User Manual.

### Check Function Keys

- 1) Press *Volume Control* to make sure sound volume can be adjusted.
- 2) Press *Touchscreen Function Control* to make sure touchscreen functionality can be enabled/ disabled. If enabled, the button will turn green.
- 3) Press *Read Light Control* to make sure the read lights can be turned on/ off.
- 4) Press *Brightness Control* to make sure the brightness of the screen can be adjusted.

### Notice:


- PCIe connection requires matching of specification. Not every PCIe card assures compatibility. Contact Advantech regional service center before use.

## (4) Optional Modules

Optional modules are hidden in several places as shown.



\*For model POC-W212-XQ1D-AXE only

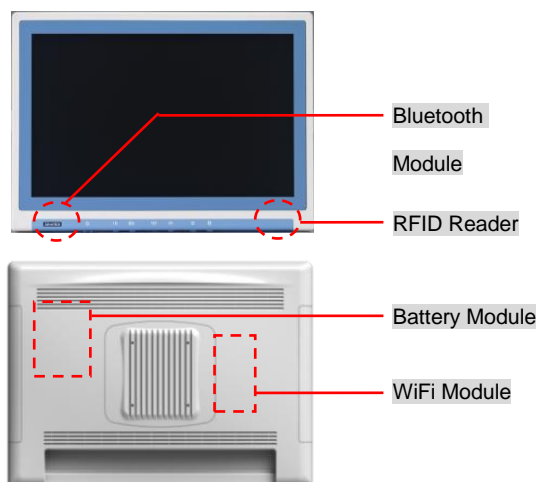

## (5) Cleaning and Disinfecting

During normal use, a POC (Point-of-Care) terminal may become dirty and should be cleaned regularly.

- 1) Prepare a cleaning agent per manufacturer's instruction or hospital protocol.
- 2) Prepare a clean cloth that has been moistened in a cleaning solution.
- 3) Wipe the POC thoroughly with a clean wipe.

### Applicable Cleaning Agents List

No	Cleaning Agents	No	Cleaning Agents
1	Cidex	11	Virkon and water (1:100)
2	Isopropyl alcohol	12	DISPATCH
3	Green tintured soap	13	SDW 70C
4	Windex	14	Trigene
5	Alcohol	15	ANTIGONE WIPES
6	Alcohol 70%	16	SporeClear
7	Incidin plus	17	AntiGone
8	Incidin liquid	18	MikroBac
9	Mikrozid liquid		
10	Chloride 1000ppm		

\* Do not immerse nor rinse a POC or its peripherals.  
 \* Do not spray cleaning agents on the chassis  
 \* Do not use disinfectants containing phenol